




Cochlear™



Supplement for N22 Implants

to the Document Pack for Cochlear™ Nucleus® CP910 & CP920 Sound Processors, CR210 Remote Control and CR230 Remote Assistant

Identifying the N22 coil

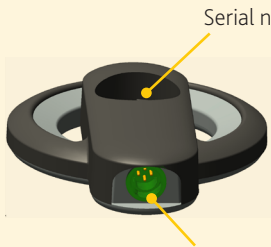

Serial number in the magnet housing (ends with "N22")

The N22 coil has a green connector for the coil cable



CAUTION

To remove your coil cable, hold the base of the cable and pull it straight from the coil - do not twist the cable.

Pairing with the CR230 Remote Assistant

Before you can start to use a CR230 Remote Assistant, you need to pair it with your sound processor.

For an N22 implant recipient, you need to turn your sound processor on, and then pair it with the remote assistant within 25 seconds.

See "Pair" in the Cochlear Nucleus CR230 Remote Assistant User Guide.



Important tips to save battery power

- **Attaching the battery module:** your sound processor **will turn on automatically** when you attach the battery module. The N22 coil will now start "searching" for your implant, which will use battery power. **Either put your coil on your implant, or turn off your sound processor to save your battery.**
- **Removing your coil from your implant:** for N22 implant recipients, your sound processor **will not turn off automatically** when off your implant. The coil will "search" for your implant, using battery power. **Whenever you intend to leave your coil off your implant, turn off your sound processor.**

Note for clinicians: The N22 coil has no telemetry, so the sound processor will not know if it is on the implant. The sound processor also will not have orange flashing lights to warn that it is off the coil. It is important that recipients always turn off their sound processor when it is off the implant to save battery power.

See "Replace the battery" and "Turn on and off" in the Cochlear Nucleus CP910 and CP920 Sound Processor User Guide.











Sound processor light indications

N22 implant recipients will not see some of the standard light indications.


Please use the table below for N22 implant recipients instead of the "Processor Lights" table in the CR230 Remote Assistant User Guide:

TYPE OF INDICATION					
	 Key Press	 Alert	 Pair Coil  Pair Wireless Acc.	 Microphone activity	 Telecoil/ Accessory/FM
MODE	DISPLAYED?				
Child	Yes	Yes *	Yes	Yes	Yes
Monitor	Yes	Yes *	Yes	No	No
Adult	No	No	No	No	No
Suspend	No	No	No	No	No

* you will not see the orange flashing light that warns you when your coil is off your implant

TIP

Set your sound processor to operate in Monitor mode.

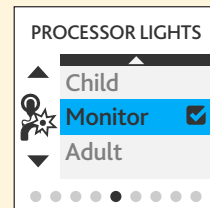

Cochlear Ltd (ABN 96 002 618 073)

1 University Avenue, Macquarie University, NSW 2109, Australia. Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Ltd (ABN 96 002 618 073)

14 Mars Road, Lane Cove, NSW 2066, Australia. Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Americas

13059 E Peakview Avenue, Centennial, CO 80111, USA. Tel: +1 303 790 9010 Fax: +1 303 792 9025

Cochlear Deutschland GmbH & Co. KG

Karl-Wiechert-Allee 76A, 30625 Hannover, Germany. Tel: +49 511 542 770 Fax: +49 511 542 7770

株式会社日本コクレア (Nihon Cochlear Co Ltd)

〒113-0033 東京都文京区本郷2-3-7 お茶の水元町ビル Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

www.cochlear.com

